



Checklist for new DASH® Rapid PCR System Customers

This checklist will help you get up and running with your DASH® Rapid PCR System. Given the similarities between the DASH® Rapid PCR System and rapid tests, many customers will be able to self-train using the provided materials.

1. Pre-Implementation

- Prepare space for your DASH® Instrument. Each instrument requires two electrical plugs (one for the instrument and one for the printer). If you have multiple DASH® instruments, provide at least 2" clearance between instruments. A power strip might be needed for multiple instruments
- Make sure you have your Wi-Fi password or ethernet cable (if using the Portal)
- Identify Portal Admin and determine staff members to train and enter into the portal
- Receive your initial order for cartridges (swabs included) and controls
- Schedule your installation and training date
- Ensure the DASH® SARS-CoV-2 & Flu A/B Test is setup in your EMR (if needed)
- [FAQs](#)

2. Reimbursement and Billing

- Provide CPT codes 87636-QW and 87631-QW to your billing team
- [LOINC codes](#)
- [CDC ICD-10 Lookup Tool](#)

3. Pre-Review Training Materials

- [Quick Reference Guide](#)
- [Instructions for Use - DASH® SARS-CoV-2 & Flu A/B Test](#)
- [Instrument Manual](#)
- [Instructions for Use - External Controls](#)
- [SDS and Other Product Documents](#)
- [Epson Thermal Printer Manual](#)

Videos

- [How to Unbox Your DASH® Rapid PCR System](#)
- [How to Run a Test Demonstration Video](#)

Training Presentation

- [DASH® Rapid PCR System Customer Training Deck - Installation](#)
- [DASH® Rapid PCR System Customer Training Deck - Portal](#)
- [DASH® Rapid PCR System Customer Training Deck - Operator](#)

Training Presentation

- [Training Certificate](#)
- [Training Log](#)
- [External OC Log](#)

Reorder Information

CN-0006 Case of 20 DASH® SARS-CoV-2 & Flu A/B Test Cartridges (includes 20 swabs)

CN-0008-CON Case of 5 Positive and Negative External controls for the DASH® SARS-CoV-2 & Flu A/B Test

CN-0012 Case of 20 DASH® Additional Sample Collection Swabs

CN-0230 Pack of 4 DASH® Printer Paper

Note: Distributor part numbers may be different. Contact your distributor sales rep to order

Contact Customer Support

Training and Installation Support: training@nuclein.com

Phone: (888) 992-3274 (DASH)

Hours: Mon-Fr 8:30am - 5:30pm CT

customersupport@nuclein.com

www.nuclein.com

www.nuclein.com/product-documents

